AMP

BUP

BZD

COC

COT

ETG

FYL

LSD

MET

OPI

THC

TML

 Salcare®

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Salcare®

Finnish rapid substance abuse home test

INSTRUCTIONS FOR USE

Read these instructions for use carefully before performing the test and keep these instructions for use for future reference.

MANUFACTURED AND MARKETED BY

SALOFA OY

Satamakatu 16, 24100 Salo, FINLAND

info@salofa.com

www.salofa.com

INTENDED USE

The Salcare® rapid substance abuse test is designed to detect the use of 12 test substances (10 drugs, alcohol plus tobacco and nicotine products) in human urine. The test is designed to distinguish a negative result from a positive result.


The test does not indicate the concentrations of the tested substances, only their presence in the sample. The test cannot measure the degree of intoxication, but only confirms the use of alcohol.

The test is not an in vitro diagnostic (IVD) medical device and is not intended for medical purposes.

It is important to consider the following regarding the rapid substance abuse test:

It is possible that an error during the test or other substances in the urine sample, not on the list of test substances, may interfere with the test and cause false positive results. For this reason, it is recommended that the municipal substance abuse service is to be contacted in order to further investigate the situation, if necessary.

TEST CARTRIDGE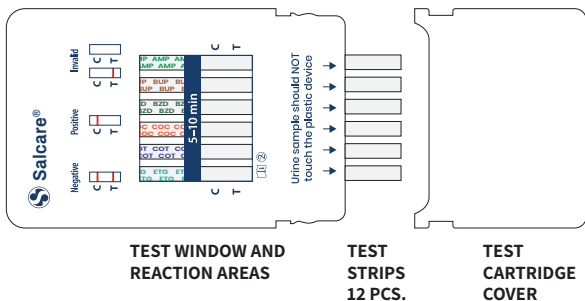

TESTED SUBSTANCES

Amphetamine	AMP	
Buprenorphine	BUP	eg. Subutex
Benzodiazepines	BZD	sedatives
Cocaine	COC	
Cotinine	COT	tobacco and nicotine products
Ethyl glucuronide	ETG	alcohol
Fentanyl	FYL	
LSD	LSD	
Methamphetamine	MET	
Opiates	OPI	
Cannabis	THC	
Tramadol	TML	painkiller

PACKAGE SIZES, PACKAGE CONTENTS AND OTHER NECESSARY EQUIPMENT

Package sizes:

- 1 test cartridge
- 3 test cartridges

Not all package sizes may be available.

The product package contains:

- Depending on the package size, 1 or 3 test cartridges in a foil pouch. In addition to the test cartridge, the foil pouch contains a desiccant packet.
- Instructions for use.

Other equipment needed to perform the test not included in the package:

- Dry and clean container for the urine sample.
- Clock or timer for timing.

PRODUCT STORAGE AND DISPOSAL

- Store at refrigerator temperature or at room temperature (+2 – +30 °C).
- Keep out of the reach and sight of children.
- The product expiration date is indicated on the outer packaging and on the foil pouch of the test cartridge.
- Used or expired products and packaging must be disposed of in accordance with waste management regulations. The product must not be disposed of in the environment.
- Cardboard packaging and instructions: energy waste. Foil pouch, test cartridge, and desiccant packet: mixed waste.

CONSIDERATIONS / PRECAUTIONS

- Do not reuse as the product is for single use only.
- The test cartridge foil pouch must not be opened before the test is carried out.
- During the test, make sure that the urine sample does not contact the plastic part of the test cartridge.
- To avoid damaging and contaminating the strips, do not touch the open ends of the strips.
- Make sure that the urine sample does not enter the test window area of the cartridge, even if splashed, as the test will no longer be usable.
- During and after the test, ensure hand hygiene and the proper cleaning or disposal of materials in contact with the urine sample.
- Do not use the test after the expiry date indicated on the product.
- Do not use the test cartridge if its foil pouch is damaged.
- Consider the completion time of the test (5–10 min) before interpreting the results.
- Read the results on both the front and back of the test cartridge.

PERFORMING THE TEST

Make sure that the test cartridge is at room temperature before performing the test.


Collect a urine sample in a dry and clean container.

The urine sample must be collected so that testing can be performed as soon as possible after collection.

Open the foil pouch of the test cartridge and remove the test cartridge from the pouch. The plastic case of the test cartridge has 12 test strips, 6 on the front of the cartridge and 6 are on the back. There is a cover on the open ends of the test strips to protect them from contamination and damage.


N.B! Do not open the foil pouch until you are ready to perform the test.

1. Remove the test cartridge cover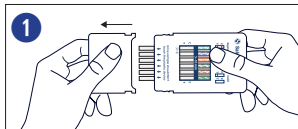


2. Hold the ends of the test strips in the urine sample for approx. 30 seconds.

N.B! Ensure that the sample does not contact the plastic part. If the sample comes into direct contact with the test window, the test is no longer usable.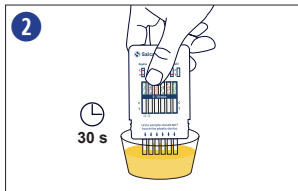


3. Replace the test cartridge cover.

Place the test cartridge on a suitable hard horizontal surface.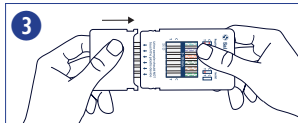

4. Read the results within 5–10 minutes of performing the test. The result is not reliably readable after 10 minutes.

There are six reaction areas on each side of the test cartridge, i.e. the test shows the results of a total of 12 different substances.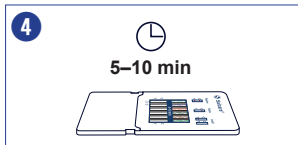

INTERPRETATION OF RESULTS


The Salcare® rapid substance abuse test provides a separate result for 12 test substances at the same time. Each test substance is interpreted as its own result based on the coloured lines formed in the reaction area of that substance. The colour intensity of the lines may vary.

After the test, a control line should be visible in the reaction area of all test substances at C. In this case, it can be confirmed that the test has been performed correctly.


NEGATIVE RESULT No substance detected in the sample

Two coloured lines appear in the reaction area of the test substance, one in the control line area (C) and the other in the test line area (T).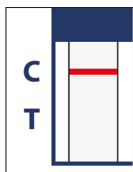

POSITIVE RESULT Test substance detected in the sample

A coloured line appears in the control line area (C) of the test substance.
No line appears in the test line area (T).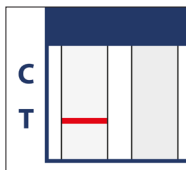


FAILED RESULT The result cannot be interpreted

If no control line (C) appears in the reaction area of the test substance, the result for that test substance cannot be interpreted.

To obtain an acceptable result, the determination of that test substance should be repeated on a new test cartridge.

EXAMPLES OF TEST RESULTS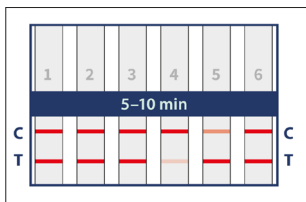


Example 1

Each reaction area shows both the control line (C) and the test line (T).

The test result for each reaction area is negative, i.e. no test substances have been detected in the sample.


The colour intensity of the lines may vary. In the figure, the test line of reaction area 4 and the control line of reaction area 5 are less visible, but if both lines are readable, the result is negative.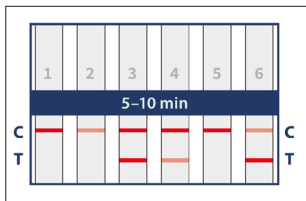

Example 2

The result of the first, second, and fifth reaction areas is positive because only the control line (C) is visible in those reaction areas.

Reaction areas 3, 4, and 6 have a negative result.

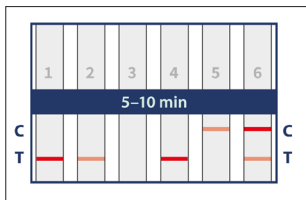Example 3

Results of reaction areas 1–4 are not interpretable because the control line (C) is not visible. The result of reaction area 5 is positive and the result of reaction area 6 is negative.

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